Breezy 100

Gebrauchsanweisung
Directions for use
Notice D'Utilisation
Gebruikershandleiding
Manuale Per l'Uso
Manual De Utilizaçao
Manual De Usuario

BREEZY

INNOVATE.
BUILD FOR THE FUTURE

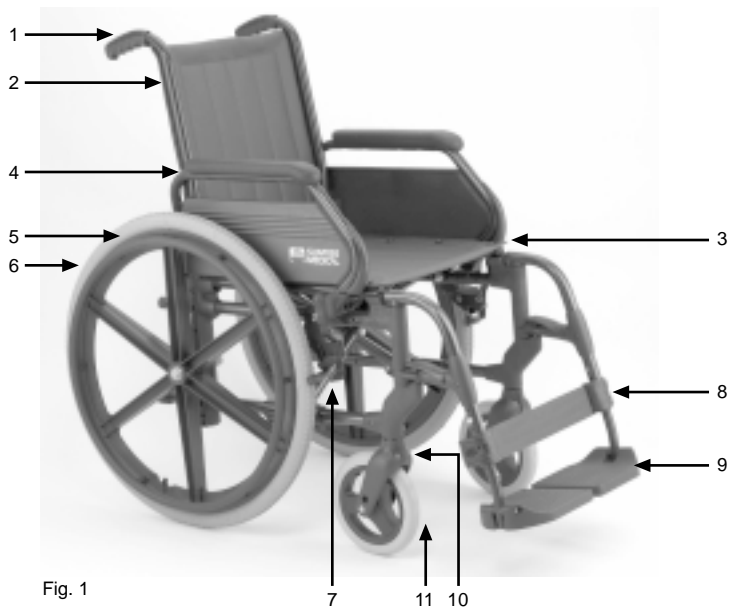
Fig. 1

Fig. 2

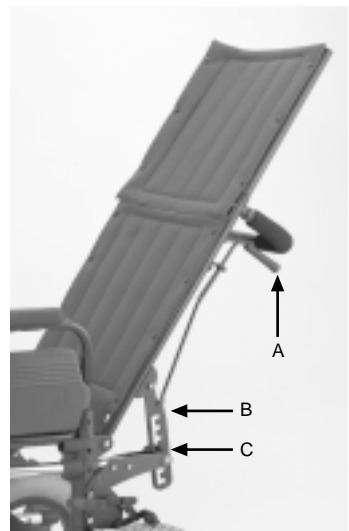
Fig. 3

Fig. 4

Fig. 5

Fig. 6

Fig. 7

Fig. 8

Fig. 9

EINLEITUNG

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INTRODUCTION

Thank you for choosing a SUNRISE MEDICAL product. In view of the broad range of components and accessories available for all SUNRISE MEDICAL wheelchairs it is in your interest to consult a health professional or authorised SUNRISE MEDICAL distributor. These wheelchairs are designed for persons with a physical handicap, to be used only for transport. These wheelchairs are suitable for use both indoors and outdoors (Class B product).

The manufacturer reserves the right to modify, without prior notice, weights, dimensions, or any other technical features described in this manual. All figures, measures, and capacities included in this manual are approximate and are not specifications.

Models: 105, 112, 115, 110, 121, 141

Dealer stamp

Sunrise Medical is ISO 9001 certified, which ensures quality at all stages of the development, and production of this wheelchair.

This product is manufactured to comply with the radio interference requirements of EEC directive 89/336/EEC Medical Device Directive 93/42/EEC.

INDEX

A. CHARACTERISTICS	9	F. CONTROLS AND MAINTENANCE	11
B. SAFETY	9	G. SPECIFICATIONS	12
C. HANDLING	10	H. MEASURES	12
D. ASSEMBLY	11	I. GUARANTEE	12
E. ADJUSTMENTS	11		

A. CHARACTERISTICS

- | | |
|-----------------------|---------------|
| 1. Handgrip | 7. Wheel lock |
| 2. Back upholstery | 8. Footrest |
| 3. Seat upholstery | 9. Footplate |
| 4. Armrest upholstery | 10. Fork |
| 5. Push rim | 11. Castor |
| 6. Propulsion wheel | |

B. SAFETY TIPS

The engineering and construction of this wheelchair have been designed to provide maximum safety. International safety standards currently in force have either been fulfilled or exceeded. Nevertheless, users may put themselves at risk by improperly using their wheelchairs. For your own safety, the following rules must absolutely be observed.

- Unprofessional or incorrect adaptation or adjustment work could increase the risk of accidents.
- As a wheelchair user, you are also part of the daily traffic on streets and pavements, just like anyone else. You should always wear clothing which is light-coloured when it is dark, so that you can be seen more easily. Also, make sure that the reflectors are not covered.
- Be careful during your first ride in this wheelchair. Get to know your wheelchair. Test what effect changing the centre of gravity would have on your wheelchair. Practice using your wheelchair on level ground as well as going uphill and downhill.
- CAUTION: The wheel locks are not intended to brake your wheelchair. They are only there to ensure that your wheelchair does not begin rolling unintentionally. When you stop on uneven ground, you should always use the wheel locks to prevent such rolling. Always use both wheel locks; otherwise, your wheelchair could tip over. The

wheel locks have not been designed to be used as brakes for a moving wheelchair. **Warning: do not put your fingers into the braking system.**

- ❑ Always make use of elevators and ramps. If these are not available, you can reach your destination with the aid of two attendants. They should grip the wheelchair at securely mounted parts. A wheelchair should never be lifted with an occupant; it should only be pushed.
- ❑ Before each use, the following should be checked: Quick-release axles (if fitted) on the rear wheels / Tyres, tyre pressure, and wheel locks. Wheel locks will not function properly if tyre pressure is too low or the interval between tyres and wheel locks is too large.
- ❑ The maximum user weight is 125 kilograms. Never overload the chair by carrying rucksacks, etc. or a child on your lap.
- ❑ Do not attempt to climb or descend gradients of more than 10% with the maximum weight of 125 kg. Always lean your body towards the top of the slope to correct for changes in the centre of gravity.
- ❑ Transportation in vehicles: This wheelchair has been crash tested in its standard configuration. However, due to safety risks Sunrise Medical do not recommend the transportation of people in their wheelchairs. Should a person need to be transported in wheelchair on transport systems, the following should be adhered to:
 1. The wheelchair must be positioned forward facing.
 2. The Unwin restraint system, model 4 point WWR/ATF/K/R or equivalent specification must be used.
 3. The restraint system must be fixed as per labelling on the wheelchair. For tie down points on the wheelchair, please look for the tie-down symbols on the wheelchair.
 4. A head restraint must be fitted and suitably positioned at all times during transportation. (Unwins HR models recommended).
 5. If possible remove seat cushion to create a lower centre of gravity.
 6. The chair is crash tested and meets the requirements of ISO 7176-19
- ❑ If the chair has been left in the sun for a long time parts such as the fabric, armrest padding and platforms may burn the skin on contact.
- ❑ It is dangerous to tip the chair back so that the front wheels are off the ground: it could overturn and injure you. Do not attempt to do "wheelies" unless you are an experienced user or are accompanied.
- ❑ When mounting kerbs and other obstacles, ensure that the anti-tipping wheels are facing backwards so that they do not become caught on the kerb.
- ❑ If the push rim is damaged replace it at once: otherwise splinters and chips could injure your hand when you push.

C. HANDLING

1. Opening: Rest both hands on each of the seat upholstery support rails and press down until the seat and back upholsteries become taut. Lower the footplates into the using position. As a safety measure, the footplates will always be in an upright position before sitting or getting out of the chair in a forward movement. **Warning: Before sitting down ensure that the seat rails are properly housed in the front structure supports. Warning: Do not place your fingers between the seat upholstery and the seat tubes when opening or folding the wheelchair. Never open the chair by pushing on the push hands.**

2. Folding: First flip up the (platform) footplate or the individual footplates. Take hold of the sling from underneath in the middle and pull upwards. The wheelchair will then fold up. To fold your wheelchair until it is as small as possible, e.g., to put into a car, you can remove the footrests (depending on the model). For this purpose, open the latch from the outside and swing the footrest to the side. Then the footrest only has to be drawn out of the frame tubing. **Warning: Do not put your fingers between the seat upholstery and the tubing of the chair when opening or closing it.**

3. Folding backrests (Ref. 115 and 110) (Fig 1)

To fold the backrest, pull upwards the locking piece (A) and then pull downwards the back rest. To assemble: Pull upwards the back rest and pull downwards the plastic piece (A) to lock it and to secure it in this position. **Warning: Be careful not to catch your fingers when folding down or raising the backrest.**

4. Reclining backrest (Refs. 121, 141) (fig.2)

Lowering the backrest: Loosen the screw knobs (C) at the bottom of the racks. Lift the two handgrips (A) upwards at the same time, while keeping the chair steady by its handles. This movement releases the lower racks (B) from their housings leaving the backrest free to be reclined. Hold the backrest firmly to prevent it from dropping suddenly. If this should happen, quickly let go off the handgrips (A) so that the racks fall into place and prevent it from falling. With the handgrips (A) in an upward position, adjust the backrest to the desired angle using the racks' notches. Once this angle

has been obtained, ensure that the racks have slipped into their notches (this operation should be done to prevent accidents caused by a sudden drop of the backrest). Release the handgrips(A). Tighten the screw at the bottom of the racks.

Raising the backrest: Proceed in the same way as for reclining the back rest.

Warning: Be careful not to catch your fingers when folding down or raising the backrest.

Warning: After the third the posterior stability of the chair is reduced, so proceed carefully.

Warning: Before raising the backrest, be sure that the user's hands or arms are not between the seat and the reclined backrest, to avoid the risk of trapping them when raising the backrest.

Warning: The reclining back is only to be used in combination with the active anti-tips setting

Warning: The reclining back is only to be used in combination with a stabilizer bar. Please always make sure that the stabilizer bar is locked in place on both sides.

Warning: Before folding the wheelchair, make sure that the stabilizer bar is removed

Warning: When adjusting the back angle, be careful not to get your fingers caught.

Warning: Before adjusting the back angle, put the anti tip tubes in to the active position

Warning: Never travel in the chair with the back-rest reclined.

5. Elevating legrest (Fig. 3) **To elevate:** Lift your lower leg and press the release button. Pull the legrest upwards and adjust it to the desired height. As soon as you release the button, the legrest will click back into place. **To lower:** Lift your lower leg and press the release button to unlock the legrest to lower it. As soon as you release the button, the legrest will click back into place.

Warning: The elevating legrest is not allowed to be adjusted into the lowest position to avoid any interference with the casters.

Warning: Keep hands clear of the adjustment mechanism between the frame and the movable parts of the legrest while elevating or lowering the legrest.

D. ASSEMBLY

1. Footrests (Fig. 4): The footrests can be rotated inwards or outwards by pressing the release latch and turning the footrest in the desired position. To remove the footrest, press the release latch, rotate the footrest approximately 90° and lift.

2. Armrests (Fig. 5): The armrest can either be flipped back or can be detached from the chair. To flip arms back, press the release latch and pull arm toward back of the wheelchair. To detach the armrest press the button located on the back of the armrest and lift it. **Warning: Make sure that the armrest is properly locked into place. Never lift the wheelchair by the armrests as these parts are detachable and lifting the wheelchair by them may cause damage to the chair and/or injury to the user. Warning: Be careful not to catch your fingers when lowering the arm-rest.**

3. Head-rest (Fig. 6): To install the head-rest, push the head-rest tubes into the tubes at the top of the back-rest until the locking pins click into position in the holes. To remove it, press the locking pins and pull the head-rest up until it comes off.

4. Reclining back-rest stabilising bar (Fig. 6): **Installation:** Insert the bar into the tubes on the back of the back-rest. Press the locking pins and hold them down while inserting the stabilising bar, until the pins lock into place in the holes in the bar. To remove the bar, reverse the process.

E. ADJUSTMENTS

1. Footrest heights (Fig. 7) To adjust the height of the footplates, loosen the retaining bolt in the frame tube, reposition footplate and tighten bolt. For these adjustment use a 10 mm. wrench. A minimum interval of 2.5 centimetres from the ground should always be maintained.

2. Wheel locks (Fig. 8) For a slight adjustment, loosen the nut of the screw on the clamp, using a 10 mm wrench. Slide the whole wheel lock forwards or backwards, through the groove of the clamp. For a larger adjustment, remove the screw and reset the wheel lock in the appropriate hole in the frame. Fix the new position with the screw and nut.

F. CONTROLS AND MAINTENANCE

MAINTENANCE: All fixings must be checked, adjusted and replaced when necessary throughout the useful lifetime of the chair. In particular, check the front and rear wheels, brakes, hangers, armrests and upholstery regularly. In addition to the maintenance and checks, general cleaning of both frame and upholstery can be undertaken. To maintain its

appearance the upholstery should be cleaned regularly. Light soiling can be removed by sponging with a mild detergent and warm water, wiping in the direction of the texture. Rinse off with clean water before allowing to dry. Do not put upholstery in a dryer. If dirt is ingrained, the surface should be scrubbed with a soft brush. The frame should be cleaned regularly with a non-abrasive cleaner followed by polishing with a non-abrasive automotive wax.

CONTROLS: A periodic check of the following procedures will help sustaining the life and efficiency of your wheelchair.

Weekly: Check de pressure of the wheels following the instructions indicated on them. Check that the cross brace's central screw is tight and in good condition / Clean the upholstery with soapy water or adequate products / Clean the metallic parts of the chair with a dry cloth or with special products for chromium plated or painted surfaces / Check the wheel locks and adjust if necessary / Check wear of front and rear tyres.

Quarterly: Check and adjust if necessary: Castor play / Play in the castor forks / Push wheels' play / Offcentering of the push wheels / Lubricate with industrial oil or grease all parts subject to friction or wear / Retighten screws and nuts / A general revision of the chair's condition, repairing whatever is necessary.

Yearly: We recommend a complete revision once a year in a specialized establishment

G. SPECIFICATIONS

Footrests: 70° Swing away, detachable and height adjusting. Elevating leg-rests optional

Armrests: Detachable and flip back

Standard Castors: 200 mm solid

Standard Rear Wheels: 600mm solid in models: 105, 115 and 121. 300 mm. solid in models 112, 110 and 141

Wheel Locks: high push-to-lock.

Spare Parts: Each wheelchair component is available as an spare part.

Accessories: **Footrests:** elevating. **Wheels:** Pneumatic tyres. **Wheel locks:** extension handles and drum brake. **Others:** Hemiplegic system, oxygen holder, belt, anti-tip tubes, table, crutch and dripping device supports... .. **Warning: Never use accessories not designed for this chair.**

H. STANDARD MEASURES

Model	Seat width	Seat height	Seat depth	Backrest height	Overall width	Overall length	Weight
105	37-46	51	41	45	55-64	105	17,4
112	37-46	51	41	45	51-60	95	16
115	37-46	51	41	45	55-64	105	17,9
110	37-46	51	41	45	51-60	95	16,5
121	37-46	51	41	75	55-64	115	20,6
141	37-46	51	41	75	51-60	106	18,6

I. GUARANTEE

SUNRISE MEDICAL guarantees this product for two years from the date indicated on the invoice, receipt or delivery note. For products containing batteries, the battery warranty is 6 months. This warranty covers parts in need of replacement, labour and transport.

In warranty claims, the user must submit the product and the purchase invoice as proof of delivery date.

This warranty does not cover:

- NORMAL WEAR AND TEAR, e.g. on upholstery, tyres, inner tubes, rims, wheels, fabric, etc.
- ACCIDENTS e.g. bent structures, scratches, flat tyres, etc.
- IMPROPER USE and damage caused by users who weigh more than the weight limit indicated in the user's manual.
- LACK OF MAINTENANCE and failure to follow the storage instructions in the user's manual.
- MODIFICATIONS or REPAIRS carried out by persons not authorised by Sunrise Medical.
- PRODUCTS WITH NO INVOICE OR SERIAL NUMBER (when appropriate).

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Sunrise Medical B.V.
Sunrise Medical Srl.